EN Sinomarin® is a 100% natural, clinically proven nasal decongestant. It is a hypertonic sea water solution (2.3% NaCl) rich with the sea's beneficial ingredients (mineral salts and trace elements). **Sinomarin®** relieves nasal congestion and sinus pressure via the natural mechanism of osmosis. Moreover, **Sinomarin®** cleanses the nasal cavities, eliminating accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents. Additionally, it helps moisturise the nasal mucosa.

The sea water used in **Sinomarin®** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality, rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Sinomarin® is free of drugs, preservatives and chemical additives.

Sinomarin® can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to improve their efficacy. As such, it helps decrease medicated treatment duration and may lead to reduced overall drug intake.

Properties:

- Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- Fluidifies mucus secretions making them easier to expel.
- Helps restore the natural function of the nasal mucosa.
- Cleanses the nasal cavities from accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents.
- Moisturizes the nasal mucosa and relieves nasal dryness.

Sinomarin® improves quality of life by alleviating symptoms linked to nasal congestion (e.g. itchy nose, sneezing, cough, sinus and ear pressure, sleep disturbance).

Sinomarin® is recommended:

- For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- In cases where medicated decongestants are contraindicated (e.g. pregnancy, breastfeeding, thyroid disease, high blood pressure, heart conditions, diabetes).
- To enhance the efficacy of medicated decongestants and to reduce their use.
- For the prophylaxis of secondary ENT infections and complications, e.g. otitis.
- To promote recovery following endonasal surgery.
- For daily hygiene (e.g. in polluted environments).
- For hydration in dry environments.

Description:

Sinomarin® Children: 100 ml nasal spray. Gentle mist spray with safety tip suitable for everyone including babies over 1 month.

*The product is also available in 50 ml & 125 ml presentations.

Sinomarin® Adults: 125 ml nasal spray. Normal stream spray for adults and children over 6 years.

Sinomarin® E.N.T.: 200 ml nasal spray. Dynamic Jet spray for adults and children over 10 years.

Composition: Sea water 70%, Purified water 30%, equivalent to 23 g/L NaCl (2.3%). Preservative-free. Drug-free. Chemical additives-free. 100% natural.

Recommended dosage:

Use as per below, or as per physician's instructions.

Sinomarin® Children: 1+ months: 1-3 sprays in each nostril daily; 6+ months: 1-2 sprays in each nostril, 2-3 times daily.

Sinomarin® Adults & Sinomarin® E.N.T.: 1 to 3 sprays in each nostril, 2 to 3 times daily.

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Precautions: For nasal use only. Do not use same nasal applicator on more than one individuals. Do not use past expiry date. Keep out of reach of children.

Advice: A mild and temporary irritation may be experienced at the beginning of use.

Storage conditions: Store below 25 °C. Pressurized container. Protect from sunlight and sources of heat. Do not expose to temperatures above 50 °C. Do not puncture or incinerate container even when empty.

Other information: Inert propellant gas (nitrogen), not in contact with solution. Solution held in a separate container.

Sinomarin® is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

Suitable during pregnancy and breastfeeding.

Sinomarin® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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Instructions for use / Gebrauchsanweisung / Kullanım talimatları

Adults & Children

- Blow nose to clear out mucus before using **Sinomarin®**.
- Break tamper-proof seal on bottle and remove protective cap.
- Place nasal applicator on top of the bottle.
- Bend neck gently forward over a wash basin and tilt head to the right.
- Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- Turn head to opposite side and repeat application in the other nostril.
- Remove nasal applicator from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- Repeat procedure if necessary.
- Remove nasal applicator from bottle, wash well with warm water and wipe dry after each use.

Babies & children who require assistance

- Wipe baby's nose clean before using **Sinomarin®**.
- Break tamper-proof seal on bottle and remove protective cap.
- Place nasal applicator on top of the bottle.
- Place baby in supine position turning its head to the right.
- Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- Turn head to opposite side and repeat application in the other nostril.
- Remove nasal applicator from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.
- Repeat procedure if necessary.
- Remove nasal applicator from bottle, wash well with warm water and wipe dry after each use.

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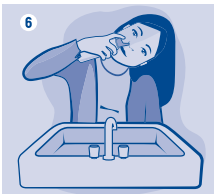
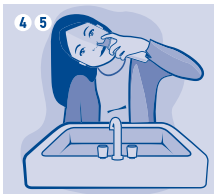
Approved 02.07.2020

Item reference code	INSERT SINOMARIN ADULTS ENG-TK-DE
Item Description	213362D
Latest update of code/ date	05/2020
Version date & time	19/6/2020 16:03
Approval by GI Regulatory/ date, name & signature	
Approval by GI Marketing/ date, name & signature	
Comments	ΤΥΠΩΝΟΥΜΕ  Pantone 280 C