**EN Sinomarin® Babies is a 100% natural, clinically proven nasal decongestant.** It is a hypertonic sea water solution (2.3% NaCl) rich with the sea's beneficial ingredients (mineral salts and trace elements). **Sinomarin® Babies** relieves nasal congestion and sinus pressure via the natural mechanism of osmosis. Moreover, **Sinomarin® Babies** cleanses the nasal cavities, eliminating accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents. Additionally, it helps moisturise the nasal mucosa.

**Sinomarin® Babies** can also be used in a **nebulized form** to help enhance respiratory function. In infants with acute bronchiolitis, nebulized **Sinomarin® Babies** may help reduce airway edema and thin airway mucus, thus decreasing airway obstruction.

The sea water used in **Sinomarin® Babies** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality, rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

**Sinomarin® Babies** is free of drugs, preservatives and chemical additives. 100% natural.

**Sinomarin® Babies** can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to improve their efficacy. As such, it helps decrease medicated treatment duration and may lead to reduced overall drug intake.

#### Properties:

- Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- Fluidifies mucus secretions making them easier to expel.
- Helps restore the natural function of the nasal mucosa.
- Cleanses the nasal cavities from accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents.
- Moisturizes the nasal mucosa and relieves nasal dryness.
- Helps alleviate symptoms linked to nasal congestion (e.g. itchy nose, sneezing, cough, sinus and ear pressure, sleep disturbance).
- In nebulized form, **Sinomarin® Babies** helps reduce airway edema and thins airway mucus, thus decreasing airway obstruction. It helps enhance respiratory function and prevent lower respiratory tract infections.

#### Sinomarin® is recommended:

- For the active removal of accumulated mucus and cleansing of the nasal cavities.
- For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- For the prevention of secondary ENT infections and complications e.g. otitis.
- For daily hygiene and prophylaxis.
- To promote recovery following endonasal surgery.
- For hydration in dry environments.
- In cases of reduced respiratory function or in acute bronchiolitis (in nebulized form).

**Description:** 18x5 ml single dose vials for infants and babies.  
\*Also available in packs containing different number of vials.

**Composition:** Sea water 70%, Purified water 30%, equivalent to 23 g/L NaCl (2.3%).

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**Recommended dosage:** A few drops in each nostril, 2 to 3 times daily, or as per physician's instructions. In cases of thick and superfluous mucus, a larger volume of product may be safely introduced in each nostril. In nebulized form, up to 5 ml 3 times daily or as per physician's instructions.

**Sinomarin® Babies** can be combined with any type of nebulizing device available in the market.

**Precautions:** For nasal and respiratory (with nebuliser) use only. Each vial is intended for single use only. Do not use past expiry date. Keep out of reach of children.

**Storage conditions:** Store below 25 °C. Protect from sunlight.

**Sinomarin®** is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

**Sinomarin®** products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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## Instructions for use / Gebrauchsanweisung / Kullanım talimatları

- 1 Wipe baby's nose clean before using **Sinomarin® Babies**.
- 2 Break-off one vial from strip.
- 3 Twist tip to open vial.
- 4 Place baby in supine position turning its head to the right.
- 5 Introduce tip of vial carefully into the left nostril and press to release the desired quantity.
- 6 Turn baby's head to opposite side and repeat application in the other nostril.
- 7 Repeat procedure if necessary.
- 8 Remove vial from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.

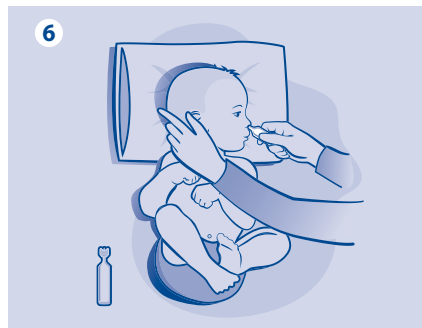
In case of use with a nebulizing device, please follow the usage instructions of the corresponding device as provided by the manufacturer.

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Approved 18.06.2020

<b>Item reference code</b>	INSERT Sinomarin Nose Care Babies 18 vials x5ml NEB Eng-Tk-De
<b>Item Description</b>	213645B
<b>Latest update of code/ date</b>	06/2018
<b>Version date &amp; time</b>	18/6/2020 13:58
<b>Approval by GI Regulatory/ date, name &amp; signature</b>	
<b>Approval by GI Marketing/ date, name &amp; signature</b>	
<b>Comments</b>	ΤΥΠΩΝΟΥΜΕ  Pantone 280 C